

 **ZERO** rock[®]
The first die stone with 0 % expansion

The reward for
the effort is
THE PERFECTION!



0 % expansion



ZERO rock[®]

The first die stone with 0 % expansion

Relaxed for the next challenge!



Perfect transfer of the mouth situation to the master model as well as to all saw models with 0% expansion.



Easy exposure of the preparation line with perfect edge stability.

Photos: Zahntechnik Jochen Graf, Mönchsau

Technical specifications

ZERO rock[®] is a special super hard stone without expansion for all saw and full models. Optimized for large-span implants and bridge works.

Mixing ratio	21 ml : 100 g
Mixing time (mechanical)	60 s
Processing time	appr. 8 minutes
Setting time	appr. 15 minutes
Hardness 24 h	> 280 N/mm ²
Hardness 30/60 min.	appr. 180/250 N/mm ²
Expansion (according to DIN)	0.00 %

Packaging unit	2x 2,5 kg	6x 2,5 kg	22 kg
ZERO rock [®] cashmere	#12061	#12062	#12065
ZERO rock [®] caramel	#12051	#12052	#12055

Your benefits

- The thixotropic consistency makes pouring easier
- The ideal final hardness for grinding
- 0.00 % expansion assures you the most precise result

**Unbeatably exact for
all precision models!**